



How to Export STL Files from CEREC Chairside Software


CEREC/ORTHO/SIRONA CONNECT:

With the release of **CEREC Ortho 1.2** software, **Dentsply Sirona** has also introduced an Open Scan Export License which enables users to export STL files from the chairside software.


SOFTWARE NEEDED:

CEREC Ortho 1.2 and higher, CEREC 4.5 and higher, or Sirona Connect 4.5. The applicable license must be activated on the License Drive which is plugged into the unit for the feature to work.

- 1. Set up a case in CEREC Software.** Capture the images, and advance to the point where the buccal bite is articulated in the Model Phase.
- 2. Click the arrow in the upper left corner of the screen. | Select Export.**

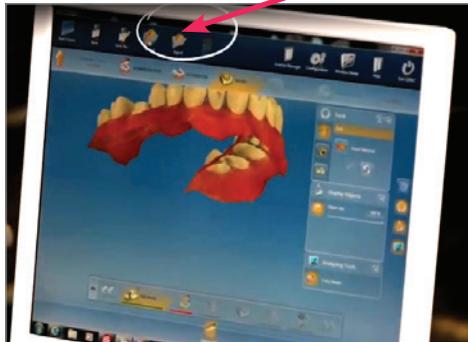


- 3. Highlight the location** where you want to save the file on the left hand side of the windows screen.

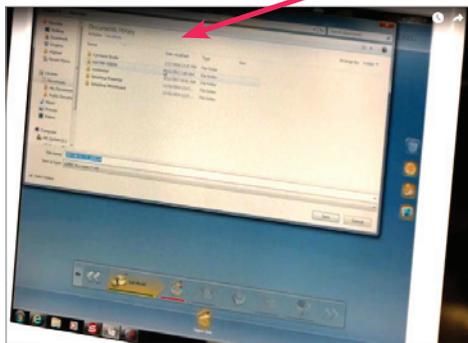



How to Export STL Files from CEREC Chairside Software (Continued)

4. Use the drop down menu in the 'Save as Type' field and **select .STL from the list**. The user can rename the file by typing whatever they choose in the File Name field. This is optional.

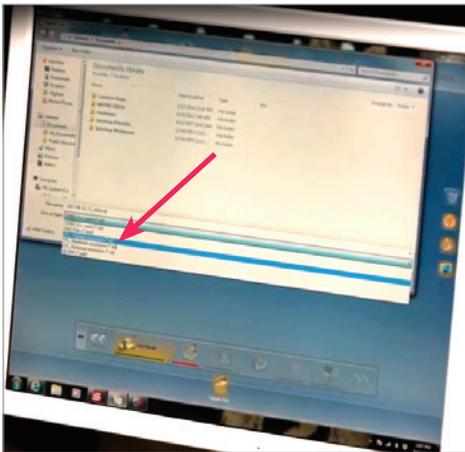

5. **Click Save.** The file is now saved in the location as directed by the user and can be copied to a flash drive or uploaded to a file sharing service as needed.


ITEMS TO NOTE:

- The Open Scan Export License is free for CEREC Club members **upon request**.
- Users must **install CEREC/Sirona Connect 4.5 or Ortho 1.2** prior to requesting the license.
- To request the Open Scan Export License, CEREC Club customers (or their branch representative) can call the distributor that sold the CEREC Scanner.
- The license will be distributed through the sironaupdates.com portal.
- Non-club customers can contact their local distributor representative to inquire about pricing and order the license.
- The Open Scan Export file type is intended for use with model printing. Users may determine if the file type is suitable for use in other formats.




How to attach files to your patient case once the scans have been exported to your desktop:

1. Login to your DDX Account

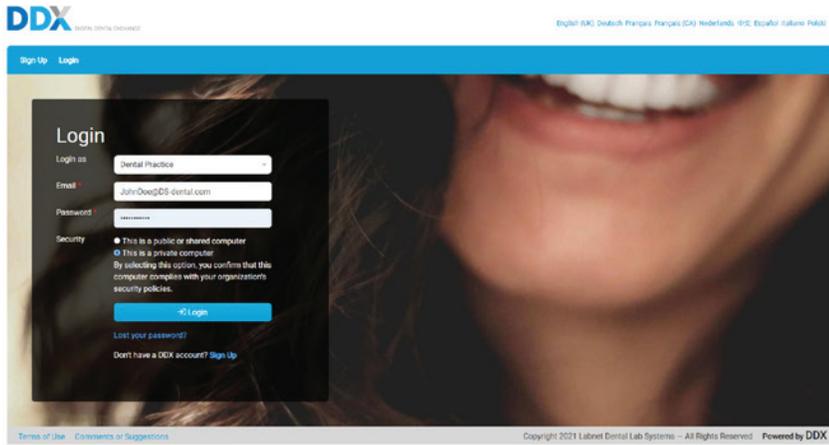

2. Select New Case

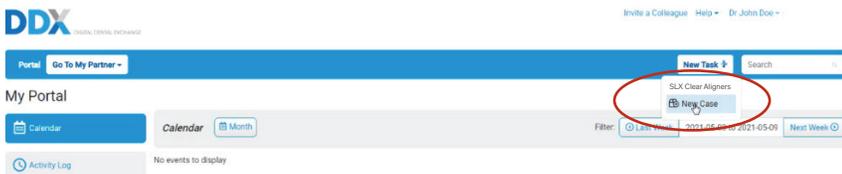

3. Enter all required patient information and details in the work request form

Portal: SLX Clear Aligner Lab - Cases - Finances - New Task - Search

Schedule a New Case

Patient Information

Provider * Dr. John Doe Gender

First Name Chart Number * 12345 Test Case

Last Name Birth date m/m/dd/yyyy

Work Requested

SLX Clear Aligners
Standard, Lite, or Express Package
Remove Procedure

Primary Concern
Chief Complaint *

Case Properties
Impression Type * Treatment Steps * Lab to Recommend *

Arch Development
Archform Width * Optimize * Midline * Optimize *

A-P Relationship

A-P relationship: Right Canine Optimize Class I * Overjet / Underbite * Optimize *

A-P relationship: Left Canine Optimize Class I * Overbite * Optimize *

A-P relationship: Right Molar Maintain * Crossbite * Optimize *

A-P relationship: Left Molar * Maintain *



4. Under Case Files, select the upload button to add scan files from your desktop folder

Enclosures

Case Files 0 up 12 Specified Files Recommended

No files uploaded

Notes

Send Date: 05/04/2021

Tags: Try to Emergency

Coupon Code:

Case for SLX Aligner Lab Uploads

Select Files to Add

DDX File Uploader - DDX - AVG Secure Browser

Open

562021-DDX aligner test

| Name | Date modified | Type | Size |
|---|------------------|-------------------|-----------|
| 562021-DDX aligner test-lowerjaw.stl | 5/6/2021 1:36 PM | Mechanism Docu... | 11,195 KB |
| 562021-DDX aligner test-occlusionfirst... | 5/6/2021 1:36 PM | Mechanism Docu... | 2,594 KB |
| 562021-DDX aligner test-occlusionsec... | 5/6/2021 1:36 PM | Mechanism Docu... | 8,005 KB |
| 562021-DDX aligner test-upperjaw.stl | 5/6/2021 1:36 PM | Mechanism Docu... | 13,252 KB |

Select Files to Add

DDX File Uploader - DDX - AVG Secure Browser

ddstaging.com

Case for SLX Aligner Lab Uploads

| | | |
|--|------------------|----------------------------|
| 562021-DDX aligner test-lowerjaw.stl 100% File Uploaded | Patient: Unknown | Specified File: [Dropdown] |
| 562021-DDX aligner test-occlusionfirst.stl 100% File Uploaded | Patient: Unknown | Specified File: [Dropdown] |
| 562021-DDX aligner test-occlusionsecond.stl 100% File Uploaded | Patient: Unknown | Specified File: [Dropdown] |
| 562021-DDX aligner test-upperjaw.stl 47.93% Uploading | Patient: Unknown | Specified File: [Dropdown] |

Select Files to Add



5. The uploaded files will appear as thumbnails under the Case Files section. Enter the notes relevant to your patient, check to accept terms & conditions before submitting your aligner case.

Case Files 12 Specified Files Recommended

562021-DDX aligner test-lowerjaw_1.stl
562021-DDX aligner test-occlusionfirst_1.stl
562021-DDX aligner test-occlusionsecond_1.stl
562021-DDX aligner test-upperjaw_1.stl

Notes

Send Date: 05/06/2021

Tags: Try-in, Emergency

Coupon Code

1000 characters remaining

I accept all Terms and Conditions

Submit Case Save Draft

Terms of Use Comments or Suggestions Copyright 2021 Labnet Dental Lab Systems – All Rights Reserved Powered by DDX